

BMI BIOMEDICAL MODELING INC.
CT SCANNING GUIDELINES for BIOMODELS

Biomedical Modeling Inc. uses advanced computer techniques to create three-dimensional polymer Biomodels which physically replicate the anatomy captured on a CT scan. BMI Biomodels are powerful tools for surgical planning and custom prosthesis fabrication. The accuracy of the biomodel depends upon the quality of the scan data. **BMI Biomodels require high spatial resolution and eliminating patient motion during the scan.** Following these guidelines is crucial to ensure that your CT data will be usable for fabricating the BMI Biomodel of your patient.

1. Settings: **Choose bone algorithm** scanning parameters for BMI Biomodels of hard tissues. Minimize artifacts in the region of interest. If you are ordering a BMI Biomodel of soft tissues, the radiologist should contact us well in advance.
2. Patient Immobilization: Any movement of the patient during the scan will distort the biomodel. If possible, the patient should be immobilized during the scan. **Do not use gantry tilt.** Following immobilization, slices should be acquired rapidly and sequentially. **The integrity of the total scan is more important than any individual slice.** Patient motion will necessitate repeating the scan.
3. Scanning Field of View (SFOV): must contain the entire region of interest with at least a 5mm margin. However, the SFOV should not be more than 10mm larger than the region of interest; in order to achieve the highest spatial resolution possible. **Caution** do not confuse SFOV with DFOV (Display Field of View).
4. Spatial Resolution: Scan slices should be as small as the radiologist feels is safe. Acquisition of **0.5mm/0.5mm** on the newest generation of MDCT is encouraged if available. If the radiologist feels that the slice thickness and spacing should be greater than 1mm x 1mm, please notify us prior to scanning, as this will affect the accuracy of the biomodel and may not meet the end use requirements.
5. Media Format: Uncompressed DICOM scan data must be sent on either CD, MOD (Magneto Optical Disk), 8mm DAT or over the internet by secure FTP. Label the media clearly with the patient name, scan date, physician name and telephone, make and model of the CT machine, scan parameters and any other pertinent information. If possible please provide a brief medical report.

The staff of Biomedical Modeling, Inc. thanks you for your attention to these guidelines. Please feel free to call Khaled at: **1-888-246-6633 (1-888-BIOMODEL)**.

Biomedical Modeling, Inc.
167 Corey Road, Suite 108
Boston, MA 02135-8214
Voice: 1-888-246-6633 or 617-738-8168
Fax: 1-888-232-9246 or 617-728-8165
khaled@biomodel.com

